

Attention

1. Place the respirator on your face as instructed and take a deep breath to check that the respirator does not leak around the edges. If it leaks, place it more rightly on your face. If you are unable to fit the respirator tightly on your face, you will not be able to protect yourself from airborne impurities with this respirator. 2. Do not wear defective respirator. Replace it if it is damaged in any way. 3. When wearing the respirator, stay away from open fire to avoid deformation of the respirator. 4. If you must remove the respirator temporarily, avoid touching its outer surface to avoid infections. 5. The respirator is disposable and may not be re-used. 6. When removing the mask, fold it and tie it together with the elastic bands.

Manufacturer:

JedX Medcare, Köysikuja 1,
01640 Vantaa, Finland
+358 20 741 5180 | www.jedx.fi



CE 2841
EN149:2001+A1:2009



1 PCS

FFP3 NR

EASY TO BREATHE

Respirator FOLDER TYPE



Elastic bands
behind the head

JEDX

Functional design for uncompromised safety

Warning

The respirator only filters the impurities listed in the instructions. The product provides no protection against gases, vapors, or oxygen shortage. Misuse may result in illness or even death. Maximum period of use 8 hours. Shelf life 5 years for date of manufacture when stored correctly. Store in a dry (RH 0...80%, -30...+40°C) well-ventilated location protected from sunlight.

Use-by date: See package.

Notified body: 2841

MNA LABORATORIES IND. TRADE. CO. LTD
Küçükbakkalköy Mah. Yenidoğan Cad.NO:21
Ataşehir/Istanbul | +90 216 574 07 08


CE 2841
EN149:2001+A1:2009

Respirator
FFP3 NR
With comfortable nose clip
Foldable

Instruction: 12-2020
Product number: 1625 W

JEDX

Product name: JedX 1625 W HL. **Package leaflet:** Folder 15,5 x 10,5 x 0,5 cm, 1 pcs / bag. **Material:** Spunbond NW, Meltblown NW. **Filtering:** solid and liquid particles; oil, dust, pollen, virus, bacterial, droplets. **Filtration standard:** EN149:2001+A1:2009, FFP3 NR, min 99 %. The declaration of conformity in accordance with the Personal Protection. Regulation can be found at www.jedx.fi

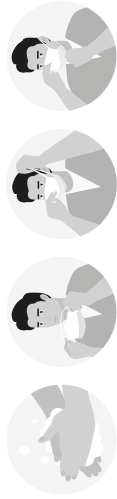

Instruction for use: 1. Wash your hands. Unfold the respirator, place the nose piece on your nose. 2. Pull the respirator over your cheeks and pull both elastic bands around your head. 3. Place the elastic bands so that the upper band sits on your head, above your ears and the lower band is on the back of your head below your ears. 4. Adjust the nose piece to fit your nose. 5. Press the nose piece with both index fingers to prevent air leakage from the edges of the respirator.

