

Attention

1. Place the respirator on your face as instructed and take a deep breath to check that the respirator does not leak around the edges. If it leaks, place it more tightly on your face. If you are unable to fit the respirator tightly on your face, you will not be able to protect yourself from airborne impurities with this respirator. 2. Do not wear defective respirator. Replace it if it is damaged in any way. 3. When wearing the respirator, stay away from open fire to avoid deformation of the respirator. 4. If you must remove the respirator temporarily, avoid touching its outer surface to avoid infections. 5. The respirator is disposable and may not be re-used. 6. When removing the mask, fold it and tie it together with the elastic bands.

Manufacturer:

JedX Medcare, Köysikuja 1,
01640 Vantaa, Finland
+358 20 741 5180 | www.jedx.fi



EASY TO BREATHE

1 PCS

FFP3 NR

CE 2841
EN149:2001+A1:2009



Respirator FFP3 NR

With comfortable nose clip

Fish type

Instruction: 12-2020

Product number: 3972 W

JEDX

Product name: JedX 3972 W FT HL. **Package leaflet:** Fish type 21 cm x 8,7 cm x 1 cm, 1 pcs / bag. **Material:** Spunbond NW, Meltblown NW. **Filtration:** solid and liquid particles: oil, dust, pollen, virus, bacterial droplets. **Filtration standard:** EN149:2001+A1:2009, FFP3 NR, min 99 %. The declaration of conformity in accordance with the Personal Protection. Regulation can be found at www.jedx.fi

Warning

The respirator only filters the impurities listed in the instructions. The product provides no protection against gases, vapors, or oxygen shortage. Misuse may result in illness or even death. Maximum period of use 8 hours. Shelf life 5 years for date of manufacture when stored correctly. Store in a dry (RH 0...80%, -30...+40°C) well-ventilated location protected from sunlight.

Use-by date: See package.

CE 2841
EN149:2001+A1:2009

Notified body: 2841
MNA LABORATORIES IND. TRADE.CO.LTD
Küçükbakkalköy Mah. Yenidoğan Cad.NO:21
Ataşehir/İstanbul | +90 216 574 07 08

JEDX

Functional design for uncompromised safety



Respirator FISH TYPE



Instruction for use:

1. Remove the respirator from the package and place it on your palm with the nose piece at the lower end of the topmost part and the elastic bands hanging down. 2. Open the respirator and place it on your face so that the nose piece fits tightly onto your nose and the lower part onto your chin. 3. Hold the respirator in place and pull both elastic bands over your head. 4. Place the elastic bands so that the upper band sits on your head, above your ears and the lower band is on the back of your head below your ears. 5. Shape and press the nose piece in position using your both hands to ensure a tight fit. 6. Check to make sure that the respirator fits tightly and does not leak around the edges or at the nose piece. If you detect any leaks, check the fit and re-shape the nose piece. If you are unable to make the respirator fit tightly, do not wear it.

